Camuvir Barriera

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Flacone da 30 ml e

Camuvir Barriera

A barrier against cold and flu viruses to prevent them developing in the nose and pharynx.

Please read the leaflet carefully before use.

When should it be used?

Camuvir Barrier is a micro gel, of botanical origin, which protects the mucus of the nose and pharynx, hindering the multiplication of cold and flu viruses **with a mechanical barrier action**. It should be used as prevention for protection or at the first signs of discomfort (burning throat, first sneeze, etc.)

How does it work?

Camuvir Barrier is a solution in the form of a micro gel based on cellulose, of botanical origin, and anthocyanins of camu camu (*Myrciaria dubia*), a fruit that grows in the Amazon forest. After spraying, it forms a film that acts as a mechanical barrier between the viruses and the mucus of the nose and mouth, preventing the proliferation of harmful organisms. Camuvir Barriera does not have any pharmacological action but only a mechanical barrier action.

When must it not be used?

In the case of known allergy to one of its components.

When must it be used only after having consulted a doctor?

In the case of frequent nose bleeds.

In the case of known pathologies of the nose, ears and oral cavity.

How do you use it?

In the pharynx. Hold the bottle and gently turn the nozzle perpendicularly. The first time, spray several times to remove the space above the liquid. Hold the bottle, aim the nozzle inside the pharynx and press the dispenser two or three times firmly with the index finger. If you want to use the product for the nose, unscrew the upper part of the dispenser and replace it with the dispenser for the nose which is supplied, screwing it in place of the nozzle for the mouth. To apply the product in the nose, insert the tube into the nose, first in the right nostril and then in the left nostril. With the bottom of the bottle in the palm of the hand, press the dispenser downwards firmly with the index finger and thumb, breathing in deeply and for the normal period of time,

without bending your head. Repeat the operation in each nostril.

How many times can it be used in a day?

If used for prevention in crowded rooms or in other situations where the risk of infection is high, use it before entering. If used at the first signs of a cold, repeat application several times a day (3-4 times) during the cold and at least for two days after the symptoms resolution.

Are there any possible unwanted effects?

During use, transitory effects such as a burning sensation, immediate but not lasting, in the mucus of the mouth and nose may occur. The sensation of burning is not harmful, but can be a sign that the product is carrying out its function. Momentary irritation of the mucus of the nose in particular can cause sneezing or an increase in nasal secretion. Use of the product should be suspended if these effects become too intense or in the case of nose bleeds.

When should a doctor be consulted?

- When the first signs of hypersensitivity appear;

- When the symptoms of a cold persist or tend to increase with a high temperature.

Warnings

The same bottle must not be used by several people at the same time, in order to avoid infection.

Keep in a cool place at a temperature of not more than 25°C.

Keep away from direct sunlight. Do not use after the date shown on the box. Keep out of the reach of children.

Avoid contact with the eyes.

INGREDIENTI INGREDIENTS:

Aqua, Myrciaria dubia fruit extract, Glycerol, Sodium Benzoate, PEG-40 Hydrogenated Castor Oil, Eucalyptus Oil, Iota-Carragenan, Sodium citrate, Thymol, Citric Acid.

CRAB
SINERGY
forniture parafarmaceutiche

Fabbricante/Manufactured by: **Crab Sinergy**

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