Zyme[®]

**A gentle Nasal Spray
augmented with Lysozyme**

What Is Zyme Nasal Spray?

Zyme Nasal spray is a medical device contains Lysozyme, Which is one of the most powerful natural antibacterial available naturally in a number of human secretions like Mucus, Saliva and Tears. Lysozyme attacks Peptidoglycans found in the cell wall of bacteria, Especially Gram positive bacteria. Zyme contains Cellulose and Isotonic saline(%0.9 NaCl) which remove and fluidify the mucus in the nasal passage of user in case of dryness of mucous membrane and cold.

How to use Zyme Nasal Spray?

Insert the nozzle in to each nostril and spray one or more puffs in to each nostril. After use, Please put the protection Cap back on the nozzle, Repeat this process according to the need. Zyme Nasal Spray contains a Safe and Natural ingredients could be suitable for long time use if necessary with Babies, Children and Adults after Doctor or Pharmacist recommendation.

Precautions should be considered?

Do not use in case of known allergy to one of product components. In case of Hypersensitivity please stop using the product and contact your Doctor. Keep out of reach of children under 3 years. Do not use after the Expiry date shown on outer box.

How to store Zyme Nasal Spray?

Please store the product under 30 °C and away from direct sunlight.

Zyme Is a 30ml Nasal Spray with a patented Nasal Applicator suitable to be used for all age groups.

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